



# Word/Bartholin Catheter

**Item # UAWC-400**

**Latex Free**



U.A. Medical Products is a Medical device distributor and can be reached at

Phone # 800-338-7788 Fax # 800-526-3441

**E-mail : sales@uamedical.com**

**Visit Our Website: [www.uamedical.com](http://www.uamedical.com)**

## Word/Bartholin Catheter

(Featuring Silicone Catheter)

U.A. Medical Products, Inc's inflatable Word/Bartholin Catheter is made of medical-grade Silicone. It is to be used for the treatment of abscesses of the Bartholin gland usually referred to as a Bartholin Cyst. The UAWC-400 catheter set contains the silicone inflatable catheter with a Scalpel, Syringe and needle.

**PRODUCT No: UAWC-400 Packaged 6 Units/BX**

### CAUTION:

Federal Law requires that a physician order this device.

### PROCEDURE:

Make an incision in the outer wall of the cyst to drain the abscess. After draining the abscess, insert the deflated silicone catheter through the incision and into the abscess. Using the 3cc syringe, filled with saline solution, carefully inflate the silicone catheter within the cyst cavity by injecting the saline into the catheter. Once inflated, remove the syringe and needle leaving the self-sealing saline inflated Word/Bartholin catheter within the abscess.

After the epithelization of a new orifice is accomplished, the saline solution and catheter are removed from the cyst. After the catheter is removed, the abscess will generally shrink and completely heal leaving the Bartholin gland to function normally.