


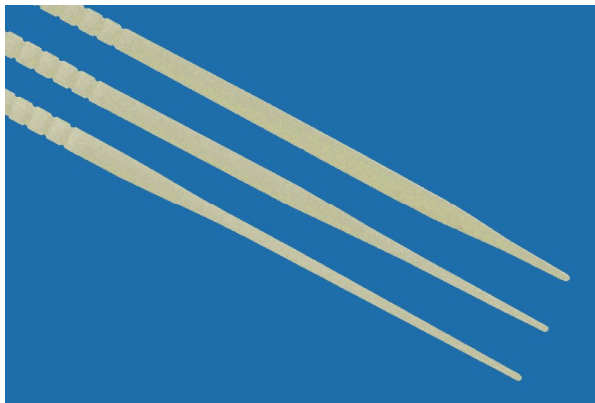

Reusable Cervical Dilators

U.A. Medical Products, Inc.
Dedicated to quality surgical Instruments...



Distributed by:
U.A. Medical Products, Inc.
P.O. Box 472205
Charlotte, NC 28247

Phone: 704-544-6862
Fax: 704-544-6864
Toll Free Phone: 800-383-7788
Toll Free Fax: 800-526-3441
www.uamedical.com

NOT MADE WITH NATURAL RUBBER LATEX

The Reusable Cervical Dilators may be reused after disinfecting. They are made of medical-grade Teflon.

PRODUCT NO. UA391-870

Reusable Type (White Color / Non Sterile)

CAUTION

Federal law restricts this device to be sold by or on the order of a physician

PRECAUTION

Check the packaging for damage for this may compromise sterility. Also, visually inspect the dilators for any surface damage.

INDICATIONS

These Cervical Dilators are especially useful in locating and dilating the external OS, the cervical canal and the internal OS. They are particularly valuable instruments to be used on post-menopausal patients with cervical stenosis.

CONTRAINDICATIONS

Cervical dilators should not be used under the following conditions:

- Acute genital track or pelvic infections.
- A pregnancy or the suspicion of a pregnancy.
- Any invasive cancer that is visible upon examination.
- Any non-compliant patient.

The Disposable and Reusable Cervical Dilators are smooth and flexible, light in weight and can be easily inserted and removed for their intended applications.

Please be aware that excessive force should be avoided and these instruments should not be used for prolonged periods of time.

They are flexible very smooth delivering excellent patient comfort for their intended indication.

THE OS FINDER

A single molded band above the tactile handle and opposite the distal end of the device identifies the disposable OS Finder and the reusable OS Finder. The OS Finder may be used to find and gently dilate the external OS. Gradual dilation in combination with its tip flexibility and smooth surface will ensure maximum patient comfort. The sound of the OS Finder is tapered 23mm to an intermediate diameter of 3.8mm.

THE CANAL LOCATOR

Two molded bands above the tactile handle identify the Canal Locator from the OS Finder and the Fundus Locator. Its sound or longer tapered tip is 45mm and extends to an intermediate diameter of 3.8mm. The Canal Locator may be used to slowly dilate the length of the cervical canal. Its flexible and smooth surface will provide maximum patient comfort.

THE FUNDUS DILATOR

The Fund Dilator has three molded bands opposite the distal tip that identifies this instrument from the others in the 3 piece OS Finder Set. The sound or tapered length from the distal tip is 85mm. The smooth and flexible Fundus Dilator is designed to pass through the entire length of the cervix to dilate the internal OS.

RECOMMENDED CLEANING AND DISINFECTING PROCEDURE FOR REUSABLE CERVICAL DILATORS

1. Cleaning: submerge in enzymatic cleaner for 20 minutes and wash with warm running tap water.
2. Disinfection: use disinfecting solution like Cidex OPA for 12 minutes

Made in Taiwan